ACCU-CHEK® FastClix®

Lancing device

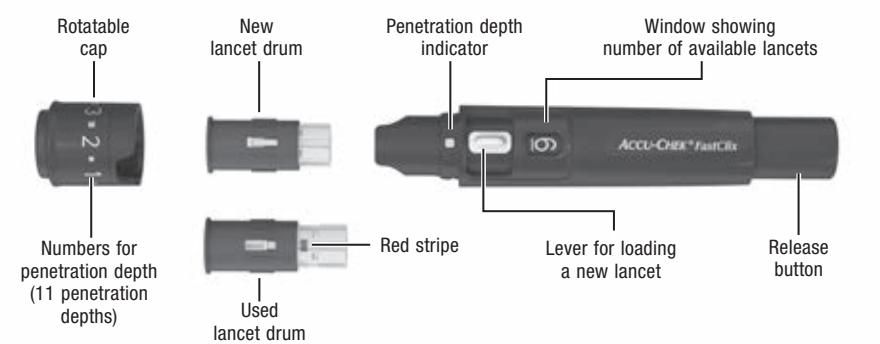
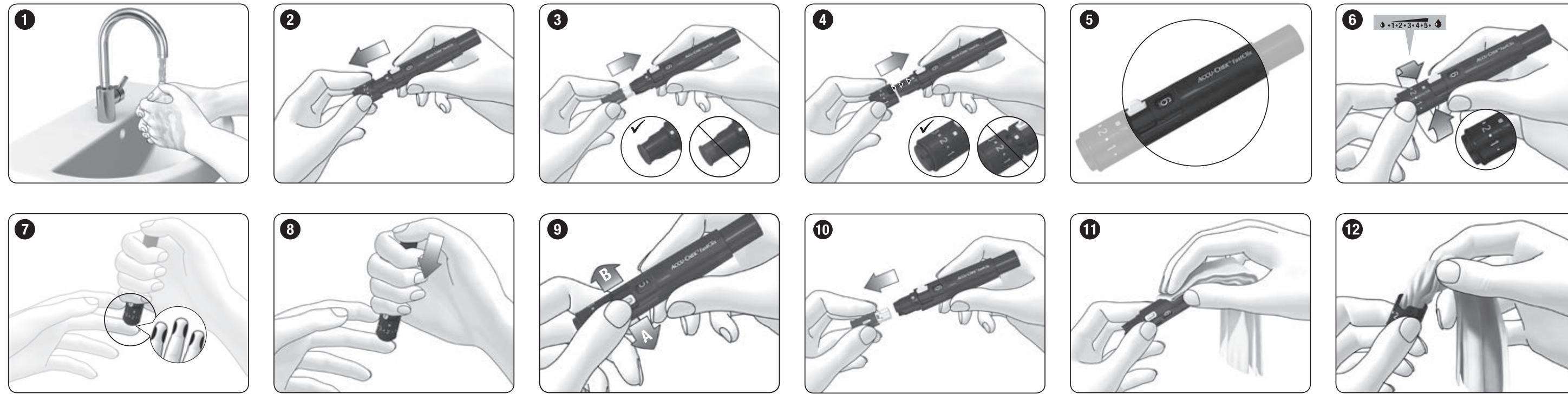
Autopiqueur

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Lancetador

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EN Instructions for use

The Accu-Chek FastClix lancing device is a device for simple, virtually pain-free collection of capillary blood from the fingertips. The lancing device has 11 different penetration depths so that you can adjust the penetration depth to the texture of your skin. With the release button, you can prime the lancing device and trigger a lancet in a single step. When a lancet is triggered, it penetrates the skin. You insert a lancet drum with 6 lancets into the lancing device. You can obtain blood using a new, sterile lancet 6 times before the lancet drum needs changing.

These instructions for use feature the following 3 symbols:

This symbol indicates a possible risk of injury or damage to your health.

This symbol draws attention to actions that could result in damage to the lancing device.

This symbol draws your attention to important information.

- The Accu-Chek FastClix lancing device is intended for personal use only!

It is only allowed to be used to obtain blood from one person.

There is a risk of infections being transmitted if the lancing device is used by other people, even by family members, or if healthcare professionals use this lancing device to obtain blood from different people. This lancing device is therefore not suitable for professional use in healthcare facilities and institutions.

• Keep the lancing device with all its components away from children under the age of 3 years. There is a risk of suffocation if small parts (e.g. caps or similar objects) are swallowed.

Only insert Accu-Chek FastClix lancet drums into the Accu-Chek FastClix lancing device. Other lancets or lancet drums may impair the operation of the lancing device or damage it permanently.

Preparation

- Wash your hands with warm water and soap before you obtain blood (see ①). Dry your hands thoroughly. This reduces contamination of the puncture site and stimulates blood flow.

Inserting the lancet drum

You have to insert a lancet drum to be able to obtain blood with the lancing device.

► Remove the cap from the lancing device (see ②).

► Insert a new lancet drum – white end first – into the lancing device (see ③).

The lancet drum is properly inserted when it is completely inside the lancing device, and only the projecting edge protrudes from the lancing device at the front (see ④).

You must not insert the lancet drum into the lancing device and simultaneously press the release button or hold the lancing device with the release button resting on a surface such as a table top. This could release a lancet and inadvertently cause injury.

► Place the cap back on again. The cut-out section at the edge of the cap must fit around the correspondingly shaped curvature with the penetration depth indicator (silver square) (see ⑤).

The cap is on correctly when it is pushed back as far as it will go, and you hear and feel it click into place.

The cap can only be correctly placed on up to the stop position when the lancet drum is completely inside the lancing device. Do not use any force to place the cap on as this might damage the cap and the lancing device. Correct the position of the lancet drum instead.

Setting the penetration depth

You can set 11 penetration depths (0.5 to 5.5). The silver square (penetration depth indicator) on the lancing device indicates the current penetration depth setting. The larger the number, the greater the penetration depth.

If you have no experience with this lancing device, we recommend setting a low penetration depth, such as penetration depth 2.

► Rotate the cap until the desired penetration depth lines up with the indicator (see ⑥).

Obtaining a blood drop

In general, blood can be obtained from any finger. Certain fingers can be unsuitable if, for example, a skin or fingernail infection is present. We recommend obtaining capillary blood from the sides of the fingertips than from the areas are the least sensitive to pain.

Always use a new lancet when you obtain blood. In doing this, you reduce the risk of infection, and obtaining blood remains virtually pain free.

• Only use the lancing device when the cap is attached. Without the cap, the lancet penetrates too deeply and the puncture can become painful.

Replacing a used lancet drum

When you have used the sixth and last lancet, you need to replace the used lancet drum with a new one.

► Remove the cap from the lancing device (see ⑦).

► Remove the lancet drum from the lancing device (see ⑧).

The window next to the lever shows the new number of available lancets, which has decreased by one, for example, 5 instead of 6.

► Proceed as in the previous step each time you need to change the lancet.

Each time a new lancet is loaded, the number of available lancets shown in the window decreases by one, for example, from 4 to 3. When you have loaded the sixth and last lancet, the window shows 1.

Replacing a used lancet drum

When you have used the sixth and last lancet, you need to replace the used lancet drum with a new one.

► Remove the cap from the lancing device (see ⑨).

► Remove the lancet drum from the lancing device (see ⑩).

The window next to the lever shows the new number of available lancets, which has decreased by one, for example, 5 instead of 6.

Setting a new lancet

If you have already used the lancing device to obtain blood, you will need a new lancet the next time you obtain blood.

► Push the lever at the side of the lancing device as far as it will go in direction A and then back in direction B (see ⑪).

The next lancet is now ready to be used.

The window next to the lever shows the new number of available lancets, which has decreased by one, for example, 5 instead of 6.

Cleaning and disinfecting the lancing device and cap

To prevent infections from being transmitted, you must clean and disinfect the lancing device and cap regularly.

• At least once a week

• When there is blood on the lancing device or cap

• Always before someone else handles the lancing device, for example, to assist you.

You need lint-free cloths in which you can wrap the whole lancing device, water and 70 % isopropanol.

Before you disinfect the lancing device and the cap, you first have to clean them to remove blood and other dirt.

► To avoid damaging the lancing device and the cap:

• Do not dip the lancing device and the cap in the fluids.

• Use only water for cleaning and only 70 % isopropanol for disinfecting.

► Remove the cap from the lancing device (see ⑫).

► Remove the lancet drum from the lancing device (see ⑬).

Cleaning

► Moisten a cloth with water.

► Wipe the lancing device and the cap thoroughly; the lancing device from the outside only (see ⑭) and the cap from the outside and the inside (see ⑮).

► Dry the lancing device and the cap with a dry cloth.

Disinfecting

► Dampen a cloth with 70 % isopropanol.

► Wipe the lancing device and the cap thoroughly for 2 minutes each; the lancing device from the outside only and the cap from the outside and the inside.

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